



DO IT AT HOME IMPRESSION KIT

We have a 99% success rate using our do it at home impression kit. You will be successful as well. If you go to the website, you can read a bit more and have all the videos you will need.



<https://www.bigearinc.com/home-impression-kit/>

<https://www.youtube.com/watch?v=dyNTlgON6zA>

PLACE ALL CONTENTS AND SHIP EVERYTHING BACK ASAP!

Keep kit at room temperature. 70 degrees.

- Fill out the invoice provided and answer the questions and check all the boxes.
- Thoroughly watch the video and understand the entire process first. Make them deep and fill the whole ear bowl. Pick up and hold the sample impression provided in the box. See how deep and completely full the ear bowl is. Make your impressions and compare them to the sample. (There is extra material in case you don't go deep enough the first time.)
- Place the person in a comfortable chair where both ears can be easily accessed.
- Take a Q-Tip and put it in your ear and show the person how deep to go. Feel that you have more than an inch to your eardrum (it is way down inside). The deeper in your ear canal the impression is, the better.
- Examine the ear canal using the ear light. Make sure the ear canal is clear of wax blockages. You may see some ear wax and some may be in the ear canal. You cannot make an impression if there is a plug of ear wax.
- Place the foam block into the ear canal. Make sure you go to the second bend. The foam block should fit snugly and fully fill the canal. Make sure the string is hanging straight down out of the ear with both strings being equal. Place block to the depth of the line on the ear light. BE GENTLE but go deep.
- Open the 2-part silicone material. VERY IMPORTANT: mix vigorously by rubbing together until it is uniform in color (about 15-20 seconds). Then roll between hands and quickly put the silicone in the syringe. If you take too long, it will set up too quickly.
- Empty syringe contents into the ear, starting deep in the ear canal, pulling out as you go and keep the tip in the material. It is not cake decorating. By keeping the tip in the material, it will smash the silicone into the ear bowl nooks. Be sure to fill the entire ear bowl with silicone.
- Silicone will be firm and set in around 4-5 minutes.
- Once firm, pull material from the ear. Start high on the ear, pull forward and twist.
- To make sure it is done properly, before you ship, text your name and a picture of both impressions to Munch at 719-930-4918 for grading.
- PLACE ALL CONTENTS BACK INTO THE BOX - EAR LIGHT; SYRINGE; COMPLETELY FILLED OUT FORM AND IMPRESSIONS.**
- PLACE SELF-ADDRESSED LABEL ONTO THE BOX AND MAIL BACK TO BIG EAR**
- Are all boxes checked?
- The time to build is approximately 6 weeks and starts when Big Ear receives your impressions. Once received, they will be put into production and they are in process. You will receive an email with a tracking number of when they will be shipped back to you.

Questions? Call or text Munch at 719-930-4918

101 Durden Park Row, Blythwood, SC 29016 USA | stonehood@bigearinc.com | 719-271-4528